



## Vista Ultrasound™

### Making Ultrasound Accessible in a Compact & Affordable Package

Natus Neurology proudly introduces Vista Ultrasound. It is an affordable, easy-to-use option that allows you to perform ultrasound studies on your EMG system hardware. This compact solution makes ultrasound readily accessible, lends efficiency to your practice and expands your testing portfolio.

The Vista Ultrasound system supports the performance of musculoskeletal and abdominal studies. It is a complementary technology to EMG studies performed in the hospital lab or private practice locations. The Vista system utilizes the USB port and display of the EDX or UltraPro S100 EMG systems with EMG v21.0.1 software or higher.

#### Vista Ultrasound includes:

- Vista application software
- 12 MHz, MV, Superficial Structures Probe: Scanning depth ranging from 0 to 2.5 cm at 8 - 15 MHz scanning frequency  
– OR –
- 7.5 MHz, VC, Deep Structures Probe : Scanning depth ranging from 0.5 to 5 cm at 5 - 12 MHz scanning frequency

#### Key Features

- Compact size allows easy accessibility wherever needed
- Overcomes space restrictions by expanding utilization of the EMG workstation.
- Affordable complementary technology
- Data quality ensures easily recognizable structures and landmarks
- Ease-of-Use allows you to save, send and print images or create full patient reports
- Built-in measurements, calculations, autoscan mode, saves time and improves efficiency
- Perform confirmatory ultrasound procedures in one patient visit

### Vista Ultrasound is now available with New EDX or UltraPro S100 systems and available separately as an upgrade

For Ultrasound on your EMG system, contact your Natus Representative or call **1-800-356-0007** or at [www.natus.com/vista](http://www.natus.com/vista)




## Vista Superficial Structures Probe


Depth Range ..... 0 - 2.5 cm  
 Pulse Frequency ..... 8 - 15 MHz  
 Scan Angle ..... 50 degrees  
 Mode ..... Full digital B Mode  
 Frame Rate ..... 12 FPS  
 Probe Lens ..... Flat  
 Cable ..... USB 2.0, 2 m (5 m opt)  
 Size ..... 15.5 cm x 3.5 cm  
 Weight ..... 100 g  
 Warranty ..... 1 year

## Vista Deep Structures Probe


Depth Range ..... 0.5 - 5 cm  
 Pulse Frequency ..... 5 - 12 MHz  
 Scan Angle ..... 60 degrees  
 Mode ..... Full digital B Mode  
 Frame Rate ..... 12 FPS  
 Probe Lens ..... Curved  
 Cable ..... USB 2.0, 2 m (5 m opt)  
 Size ..... 17 cm x 3.5 cm  
 Weight ..... 100 g  
 Warranty ..... 1 year

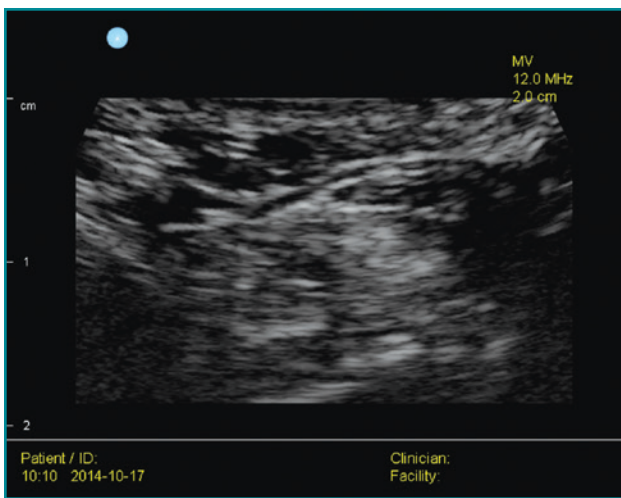

The Image above is the Carpal Tunnel taken with the Vista Superficial Structures Probe

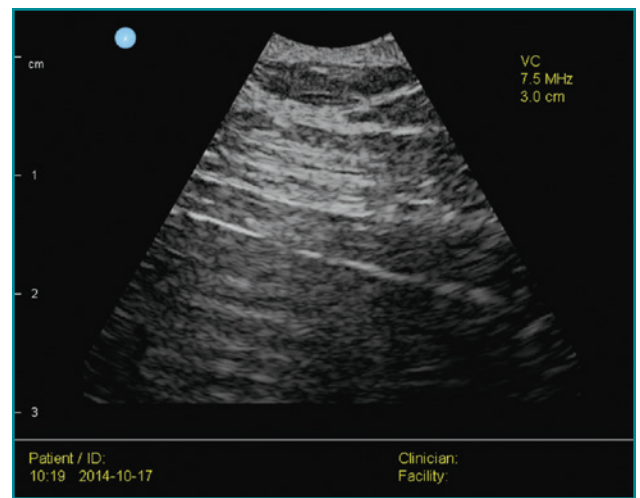

The Image above is the Diaphragm taken with the Vista Deep Structures Probe

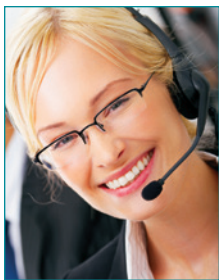

## Service

Natus Neurology is committed to providing exemplary service to our customers. Our dedicated and experienced Customer Service Team will assist with every aspect of an order. To support our products, we provide factory-trained Field Technicians and Clinical Application Specialists for onsite support. Additionally, we provide an in-house Technical Support Team, staffed with experts, and a strong distribution network in International Markets to offer a wide range of service options. Allowing our customers more time to care for their patients is our goal. Customer loyalty is our reward.

## Supplies

Natus Neurology offers a full range of neurodiagnostic accessories and supplies promoting patient comfort. Our dedicated customer service team provides a streamlined order and shipping process to save you time and money.

To learn more about Natus Neurology Service Programs or our full line of Supplies and Accessories, contact your local distributor or sales representative.  
 US Customers Call: **1-800-356-0007**



### Natus Neurology Incorporated

3150 Pleasant View Road  
 Middleton, WI 53562 USA  
 Tel: 1-800-356-0007  
 1-608-829-8500  
 Fax: 1-608-829-8709  
[www.natus.com](http://www.natus.com)